

## G-EYE™ System

*The smart innovation in enhancing detection capabilities  
powered by SMART Medical Systems.*



# A new era in colonoscopy.

Straightening intestinal folds.




A new era in colonoscopy starts with the G-EYE™ HD+ endoscopes, featuring a permanently integrated latex-free balloon at the bending section for advanced detection and therapy.

Employing a Unique Controlled Withdrawal™ technique with the balloon partially inflated, the G-EYE™ HD+ endoscope straightens and stretches the colonic folds. It supports physicians to find potential adenomas,

flat lesions and polyps hiding behind, between or on the colonic folds. As needed, the balloon can be fully inflated to anchor and stabilise the endoscope during intervention and especially during polypectomy in difficult positions.

## Key benefits of the G-EYE™ System

1. **Increased detection rate** by straightening colon folds, centralizing the endoscope optics and eliminating bowl slippage.
2. **Improved therapeutic capabilities** due to colonoscope stabilization which results in faster and more controlled procedures.
3. **On-demand two balloon endoscopy** by using NaviAid AB disposables for small intestine procedures.

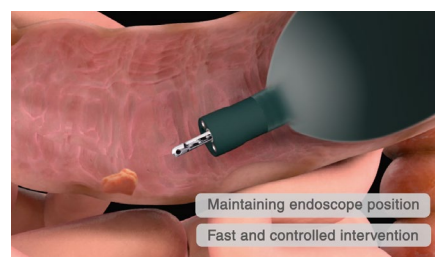

## G-EYE™ HD+ Video Endoscopes:

Product	Insertion tube [ø mm]	Distal end [ø mm]	Instrument channel [ø mm]	Working length [mm]	Field of view [°]	Tip deflection up/down [°]	Tip deflection right/left [°]	Inflated balloon diameter [mm]	Remarks
G-EYE34-i10L/F	11.6	11.5	3.8	1,700/1,500	140	180/180	160/160	Up to 60	HD+, Water Jet System, permanently integrated reusable balloon
G-EYE38-i10L/F	13.2	13.2	3.8	1,700/1,500	140	180/180	160/160	Up to 60	HD+, Water Jet System, permanently integrated reusable balloon
G-EYE38-i10F2	13.2	13.2	3.8	1,500	140	180/180	160/160	Up to 60	HD+, Water Jet System, increased rigidity, permanently integrated reusable balloon

## Constant monitoring

SPARK<sup>2</sup>C inflation system ensures constant monitoring and leak testing before and during the entire procedure. It provides controlled air pressure in three intermediate levels to one or two balloons. For accurate leak detections, a separate automatic hand held leak tester will be available to be used anytime during reprocessing in both pressure and vacuum states.

The G-EYE™ HD+ system complies with the EU MDD (CE 0482).



Leak Tester

## Advantages of SPARK<sup>2</sup>C (inflation system)

- Constant monitoring and leak testing before and during the entire procedure
- Anchoring pressure for stabilisation
- Unique Controlled Withdrawal™ pressure
- Easy operation



SPARK<sup>2</sup>C

## SPARK<sup>2</sup>C (inflation system)

Electrical input voltage [VAC]	Electrical input frequency [Hz]	Dimensions [mm]	Weight [Kg]	Set pressure tolerance [mbar]	Inflated balloon set pressure [mbar]
100-240	50-60	280 x 195 x 90	1.3	±10	Anchoring pressure 70 mbar. Unique Controlled Withdrawal™ pressure – 3 intermediate levels

## On demand enteroscopy at any time

The G-EYE™ HD+ endoscope, combined with the disposable AB (Advancing Balloon) applied through the instrument channel, provides an ultimate assembly for performing deep and small intestine two-balloon endoscopy.

The G-EYE™ HD+ endoscope as well as the AB devices are controlled simultaneously by the SPARK<sup>2</sup>C inflation system.



AB disposable

**EMEA Headquarter****Germany**

PENTAX Europe GmbH  
Julius-Vosseler-Straße 104  
22527 Hamburg  
Tel.: +49 40 / 5 61 92 - 0  
Fax: +49 40 / 5 60 42 13  
E-mail: info.emea@pentaxmedical.com

**United Kingdom**

PENTAX U.K. Limited  
PENTAX House  
Heron Drive, Langley  
Slough SL3 8PN  
Tel.: +44 17 53 / 79 27 33  
Fax: +44 17 53 / 79 27 94  
E-mail: medical.uk@pentaxmedical.com

**France**

PENTAX France Life Care S.A.S.  
116 quai de Bezons  
B.P. 204  
95106 ARGENTEUIL CEDEX  
Tel.: +33 1 / 3025 7575  
Fax: +33 1 / 3025 7445  
E-mail: clients.fr@pentaxmedical.com

**Netherlands**

PENTAX Nederland B.V.  
Edisonring 6  
6669NB Dodewaard  
Tel.: +31 88 / 5 30 30 30  
Fax: +31 84 / 210 51 44  
E-mail: info.nl@pentaxmedical.com

**Italy**

PENTAX Italia S.r.l.  
Via Dione Cassio, 15  
20138 Milano  
Tel.: +39 / 02 50 99 58 1  
Fax: +39 / 02 50 99 58 60  
E-mail: marketing.lifecare@pentaxitalia.it

**Spain**

SIMMEDICA – Sistemas Integrales  
de Medicina, S.A.  
Avenida del Sistema Solar 25  
28830 San Fernando de Henares · Madrid  
Tel.: +34 91 / 301 62 40  
Fax: +34 91 / 751 31 15  
E-mail: sim@simmedica.com

**Russia**

Moscow Representative office  
of PENTAX Europe GmbH (Germany)  
1-ya Tverskaya-Yamskaya ulitsa, 25,  
stroenie 1, 6th floor  
125047, Moscow  
Tel.: +7 495 995 10 91  
Fax: +7 495 995 10 91 (fax ext. 234)

**Turkey**

PENTAX Turkey  
Veko Giz Plaza, Meydan Sokak  
No:3/43 343396  
Maslak – Istanbul  
Tel.: +90 212 / 705 05 26  
Fax: +90 212 / 705 05 00

EC	REP
----	-----

**MedNet GmbH**

Barl Straße 10  
48163 Münster  
Deutschland

**smart**<sup>o</sup>  
medical systems

**Manufacturer:**

Smart Medical Systems Ltd.  
10 Hayetsira St.  
43663 Ra'anana · Israel  
Tel: +972 9 / 7444 321  
Fax: +972 9 / 7444 543  
E-mail: info@smartmedsys.com

